
II. Abutment Level Prosthesis


1. Solid Abutment & Components

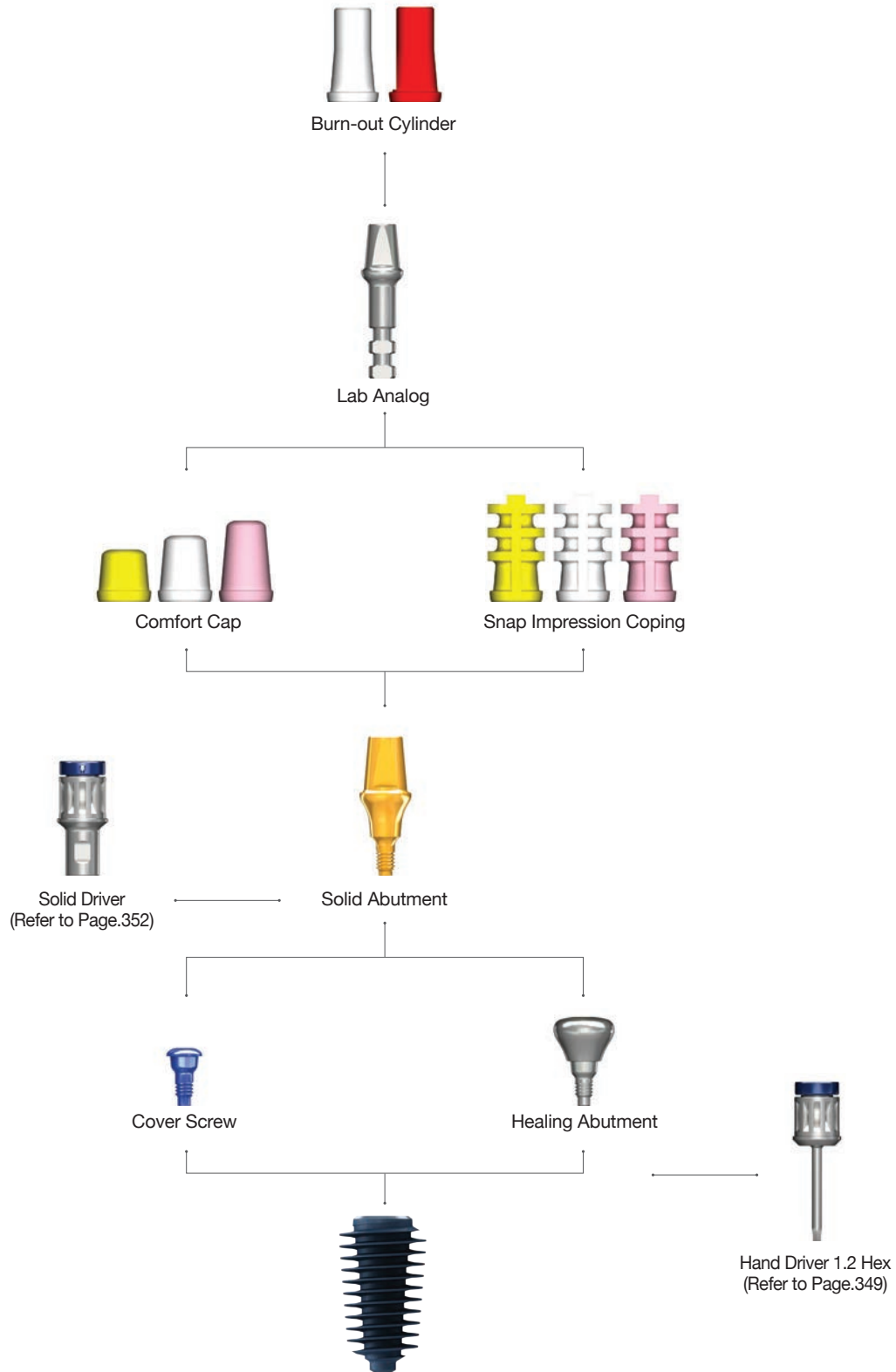➔ Solid Abutment Option

Solid Abutment

- Used in cement retained restoration only.
- Solid Abutment should be placed into patient's mouth before taking impression.
- Onebody (screw + abutment)
- Should be tightened with a Solid Driver and a Torque Wrench : 35Ncm
- Four different profile diameters. (Ø4.0/5.0/6.0/7.0)
 - Should be tightened with special Solid Driver.
 - Wider profile has bigger post angulation. (4mm - 8°, 5mm - 10°, 6mm - 12°, 7mm - 14°)
- Four different cuff heights. (2/3/4/5mm)
- Three different post heights. (4/5.5/7mm)
- Recommend torque : 35Ncm

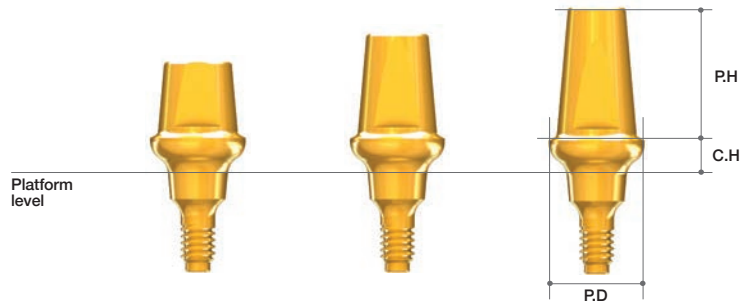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
➔ Components for Solid Abutment (Continued)

Snap Impression Coping

- For impression on Solid Abutments.
- 3 colors for different post heights.
- 4 different diameters for profile diameters. (Ø4, 5, 6, 7)
- Do not use when abutment is trimmed.

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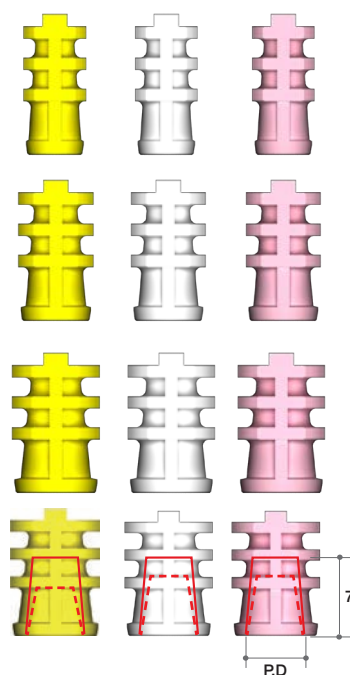
Comfort Cap

- Protects the Solid Abutment and minimizes irritation to tongue and oral mucosa.
- Can be applied under temporary prosthetics.
- Color coded according to post heights.

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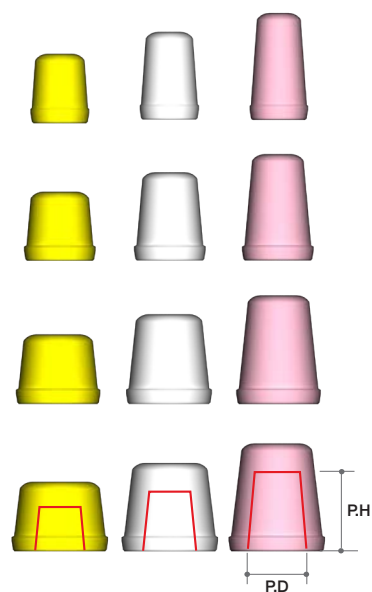
➔ Components for Solid Abutment

Lab Analog

- Directly connected to the Snap Impression Coping in the impression to make a stone model.

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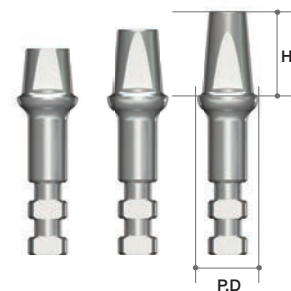Burn-out Cylinder

- Fits with a Lab Analog(solid level).
- Easy to wax-up and accurate casting.
- White Cylinder for multiple unit.
- Red Cylinder for single crown.

| Profile Diameter | Type | Ref.C |
|------------------|----------|-----------|
| Ø4.0 | Multiple | AANBCB470 |
| Ø5.0 | | AANBCB570 |
| Ø6.0 | | AANBCB670 |
| Ø7.0 | | AANBCB770 |
| Ø4.0 | Single | AANBCS470 |
| Ø5.0 | | AANBCS570 |
| Ø6.0 | | AANBCS670 |
| Ø7.0 | | AANBCS770 |